

## Advice on douche specification

There are 3 douche arm operations available.

### Standard

- For anal cleansing after defecation.
- Suitable for male and female.

### Single Extended Spray

- Normally for gynaecological use in hospitals.
- Can be specified when a client cannot sit far enough back for effective anal cleansing with standard douche.
- Required with certain support systems, please contact our technical department for advice.
- Can be used for elderly female urinary incontinence.

### Extended Twin Spray

- Only suitable for females who desire to wash front and back simultaneously.

1 TWIN SPRAY IS NOT SUITABLE AND SHOULD NOT BE SPECIFIED FOR MALE USE.

- 2 With twin spray the volume of warm washing water is significantly decreased i.e. warm wash will not last as long.

3 The purpose of twin spray is not to wash twice as well nor to wash a larger area and should not be specified for all female users unless there is a specific requirement as described above.

4 The front wash of single and twin extended douche arms is angled back at 15° from vertical to effectively wash the vaginal area and to avoid faecal contamination of the perineum. The drying function of the Closomat is significantly compromised (the drying air is discharged from the back of the pan i.e. directed towards the front).

5 Temperature and pressure of the wash are both reduced for washing the front of the body, this reduces effectiveness of anal cleansing.

**Where anal cleansing is the prime reason for deciding to buy a wash & dry (shower) toilet then standard douche should be specified.**

### Closomat douche showing 3 positions

